EN

Name
ALHYDRAN with high sun protection
Product description
ALHYDRAN with sun protection is a medical hydrating cream offering high-level protection against UVA and UVB rays (SPF 30). ALHYDRAN provides optimal and long-lasting hydration and prevents moisture loss. Ideal for treatment of disrupted skin barrier. Suitable for (after)care of burn wounds, treatment and prevention of scars, treatment of (extremely) dry skin and reduction of redness and itching.

ALHYDRAN is hypoallergenic and contains no parabens or fragrances. Suitable for all skin types.

Ingredients
Aloe Barbadensis Leaf Juice (Aloe Vera Gel from Aruba), Aqua (Water), Caprylic/Capric Triglyceride, Zinc Oxide, Homosalat, Octocrylene, Glyceril Stearate, Cetearyl Alcohol, Decyl Oleate, Cetearyl Isononanoate, Polysorbate 60, Ethylhexyl Salicylate, Butyl Methoxy-dibenzoylmethane, Simmondsia Chinensis Seed Oil (Jojoba Oil), Sorbitan Stearate, Ceteareth-20, Propylene Glycol, Ceteareth-12, Cetyl Palmitate, Phenoxethanol, Jojoba Esters, Caprylyl Glycol, Chlorphenesin, Polyhydroxystearic Acid, Tocopheryl Acetate (Vitamin E Acetate), Ascorbic Acid (Vitamin C), Disodium EDTA.

Properties
ALHYDRAN is a medical cream with a base of freshly manufactured Aloe Vera gel. Aloe Vera gives the cream strong moisturising properties and a mildness particularly suitable for damaged and vulnerable skin. The other ingredients of the ALHYDRAN product line contain a high level of high-end cosmetic oils and fats, which further contribute to the general moisturising characteristics of the product. ALHYDRAN makes the skin supple and elastic, and is particularly suitable for skin types with reduced elasticity. The product is extremely mild and tolerated by a wide range of skin types.
High protection
SPF 30
UVB + UVA
Paraben free
Fragrance free

High sun protection factor, SPF 30
The sun's rays are healthy for humans, but also potentially dangerous. There are two types of sunrays: UVB and UVA. (Extensive) exposure to UVB rays can lead to sunburnt skin. (Extensive) exposure to UVA rays accelerates skin ageing and can result in over-sensitivity to sunlight and hyperpigmentation (pregnancy mask, other discolouration). Both types of rays can affect cell DNA and, in the long term, contribute to the development of skin cancer. ALHYDRAN with sun protection has an SPF factor of 30 and protects against UVB as well as UVA rays.

Indications/uses
- As a supplement to standard ALHYDRAN, to protect the skin against damaging UV rays. Routine application is advised if the skin barrier is disrupted.
- Aftercare of recently healed skin following acute and chronic wounds.
- Aftercare of damaged skin, in order to prevent scarring.
- Improves the condition of existing scars resulting from burn wounds.
- Prevents and reduces itching and pain immediately after skin has healed closed.
- Reduces eczema symptoms.
- Aftercare of burns, in particular skin care, immediately before and after wearing pressure garments and/or silicone dressings.
- Aftercare of wounds caused by freezing.
- Preventive skin care during oncological and radiation therapy.
- Aftercare of skin grafts and skin transplantation donor sites.
- Preventive care of red/dry skin and stretch marks.
- Aftercare following chemical peeling and laser treatment.
- Aftercare following medical and cosmetic surgery.

How to use
- Apply well in advance.
- Apply liberally. About 6 teaspoons are required to fully protect an adult body.
- Regularly reapply (every 2 hours). Especially after swimming, exercising and towel drying.
- Avoid strong, midday sunlight (between 11:00 and 15:00 hours).
- No product provides 100% protection; so do not stay in the sun for too long. Not possible? Then wear clothing for extra protection: hat, t-shirt and sunglasses.
- Always keep babies and young children out of full sunlight.

Do not use
- If there is an allergic reaction to one of the ingredients.
- On open wounds, or skin perforated by stitches that has not yet healed.
- On infected skin.
- On chronic rash or other irritated skin.
- Do not use on children under the age of 3 months.

Special precautions
- Keep out of reach of children.
- External use only.
- Use only on healed, closed skin and not on open wounds.
- Avoid direct contact with eyes and mucous membranes.
- Do not use if packaging has been damaged or opened in transit.
- When using on children under 6 months, please consult your physician first

Side-effects
There are no known side effects to date. Any serious incidents that have occurred in relation to the device should be reported to the manufacturer and the competent authority (of the member state) in which the user and/or patient is established.

Storage
- See pack for expiry date.
- Store at temperatures not exceeding 25°C.
- The product may slightly discolour or change consistency after opening due to the high concentration of natural ingredients - which include fresh, Aloe Vera gel. This does not affect the proper functioning of the cream.

Date of last revision of instructions for use
November 2024

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